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Instructions for use

Read carefully before first use!

In the event of cardiac arrest, survival and the prevention of long-term neurological damage largely depend on immediate resuscitation measures such as first aid being taken. The REANIMATOR® has been developed to prevent people without medical knowledge from being afraid to carry out urgent resuscitation measures and to provide them with an easy-to-use hand-pressure cardiac massage aid. The device has a purely mechanical structure and does not contain electronic components in such a way that the REANIMATOR® can be used at any time immediately.

I.- Purpose of use

The REANIMATOR® is a practical and simple device that is used by inexperienced people to help in an emergency to start the necessary resuscitation measures in another person. In emergencies, a serious problem that often occurs is that people without knowledge of medicine they hesitate before applying the first aid measures that are necessary. The reasons can be very varied: little knowledge and experience in resuscitation, spontaneous feeling of helplessness and fear of acting badly or picky when in direct contact with the patient's body. But, even if the inexperienced person begins resuscitative measures without qualms, there is great uncertainty about how hard to press on the rib cage and how often. It is also often not clear at what exact point cardiac massage should be applied. The REANIMATOR® takes all these worries and thoughts away from its user. Another advantage is that direct contact with the body is avoided.

II.- Description of the REANIMATOR®

The REANIMATOR® has very compact dimensions, 110 mm x 170 mm, and weighs only 130 grams and is stored in a transparent – green plastic case in a hygienic and airtight manner. The REANIMATOR® is a purely mechanical device that works without a power supply. nor electronic components. Therefore, the REANIMATOR® can be used at any time without further requirements. Pay attention to the fact that the REANIMATOR® is a single-use product that once applied in an emergency cannot be reused. The central component of the REANIMATOR® is the white, stable base plate made of -PE plastic, on the underside of which is laid a layer of body-compatible, dark gray polyurethane foam for homogeneous pressure distribution and fluid absorption. The external shape of the REANIMATOR® facilitates its correct positioning on the chest. On the base plate there is a complex arrangement of coil springs that transmit the pressure force to the thorax. With a force of 410+/- 30N, which corresponds to a compression level on the rib cage of 50mm to 60mm. A “tic” sound is generated that is clearly audible. The REANIMATOR® is topped by a round upper part with a red mark that serves as a pressure surface when using the device. On this pressure surface, the REANIMATOR®'s method of use is symbolized through pictograms that are understandable throughout the world, in such a way that in case of emergency, it is enough to take a look at it to make safe use of it. However, it is required that you have read and understood these previously detailed instructions for use.

III.- Operation

The external pressure in cardiac massage should be approx. 410 N. What to an inexperienced person may seem like a lot. But only in this way can the necessary level of pressure in the ribcage, which ranges from 50 mm to 60 mm, be achieved. A special spring in the REANIMATOR® signals that the correct pressing force of 410 N has been reached with a clear metallic “tic” sound. A loud “tac” sound is also heard on the discharge of pressure. The continuous change of the pressure “tic” and the shock “tac” helps the user to determine the correct rate of change between the active application of pressure with shocks and shocks. With this rhythm indicator, the person in charge of first aid achieves the necessary pressure frequency of between 100 and 120 compressions per minute. The special shape of the REANIMATOR® facilitates its correct positioning in the lower third of the patient's sternum. In this way, the pressure of the cardiac muscle can be perfectly transmitted inside the bony thorax.

IV.- Instructions for use

General procedure for starting resuscitation measures with a person with whom you cannot speak:

- Try to talk to the patient and shake his shoulders.
- Ask for help by shouting loudly, draw the attention of other people to the situation and order a call to the emergency number 112.
- Release the patient's airways by carefully lifting the chin and extending the head backwards.
- Check if the patient is breathing by looking at him, listening to him or touching him.
- If you do not detect normal breathing, you do not see the ribcage rise and fall or you do not perceive other vital signs, you should start immediately with the cardiac pressure massage.

To begin, take the REANIMATOR®, pull up the upper lid to open the green-transparent plastic box and remove the device. Free the patient's ribcage from any clothing up to the chest and abdomen. Kneel facing the patient at chest height next to him. Place the REANIMATOR® foam side down directly on the skin at the level of the sternum so that the pointed end of the REANIMATOR® is pointing at the patient's feet and approximately coincides with the tip of the sternum. The entire lifting force is deployed a little higher, as appropriate (see drawing).

Place the hands one on top of the other on the pressure surface marked green and begin the resuscitation with arms outstretched. In addition, you must exert such great pressure on the REANIMATOR® by means of blows that the loud “tic” sound is heard. Then quickly release the pressure and right after the “tac” sound, apply full pressure again. The sequence of pressure strokes and discharges should be as rapid in succession as the “tic-tock” sounds that indicate the rhythm. Think of the strokes of a clock hand. It must reach a frequency of 100 to 120 blows per minute. Your arms should always remain extended so that the pressure force comes from the shoulders and trunk.

You should continue this resuscitation until the first clear vital signs are felt or until the other resuscitative measures can be carried out by healthcare personnel or the emergency physician. Here you can once again see in summary the steps to follow for the use of the REANIMATOR® for cardiac pressure massage:

- Position the REANIMATOR® on the sternum, in the lower third.
- Pay attention to place the hands on the pressure surface of the RESANIMATOR in such a way that the arms are extended.
- The pressure stroke must always be exerted with such force that the "tic" is heard.
- The alternating rhythm of pressure and load is indicated by the "tic-tac".
- Continue cardiac massage with a minimum pressure rate of 100 per minute until clear vital signs are detected again or until the patient can be taken care of by the emergency physician.

V.- Limits

The REANIMATOR® is not a substitute in any way for the necessary medical-professional treatment or care by healthcare personnel and emergency physicians. Therefore, it is absolutely necessary that you continue to call for help even after starting resuscitation measures. The REANIMATOR® is only an auxiliary means to take advantage of the valuable time until the arrival of rescue personnel by applying the first resuscitation measures. In no case can it be guaranteed that these resuscitation measures will work. After all, this depends on the state of health of the patient.

VI.- Warnings

Use the REANIMATOR® only for its intended use and never for any other purpose, handle the device with care and store it in the closed and protected external box in a clean environment and out of the reach of children. Avoid the effects of frost and intense heat. In no case should you open or disassemble the REANIMATOR®. Check regularly (recommended every 6 months) if the REANIMATOR® works without problems. In this context, you should also read these instructions for use every time you test it in order to safely master the use of the REANIMATOR® even in stressful situations. For the routine check of the operation described in the instructions, press the REANIMATOR® a few times with force until you hear the characteristic “Tic-Tac” when pressing and releasing. If the REANIMATOR® has been dropped or otherwise damaged, it must be replaced with a new device, as the springs inside and in the correct position are no longer guaranteed. Pay attention to the expiration date of its use preferably, which appears on the white basic plate of the REANIMATOR® and indicates the month and year. After this date, you must replace the REANIMATOR® with a new device. As soon as the REANIMATOR® has been used once in an emergency to carry out resuscitation measures, it may not be used again under any circumstances. This is a single use device! The REANIMATOR® can be disposed of together with the plastics, as the device has a purely mechanical structure and does not contain any electronic components.

The REANIMATOR® cannot be used on people under 16 years of age! Special caution is required when using the REANIMATOR® on frail or elderly people. Do not use the REANIMATOR® near a magnetic resonance tomograph (MRT).

In case of use with a defibrillator, pay attention to the manufacturer's safety instructions. Do not use the REANIMATOR® on people with normal vital signs!

VII.- Risks

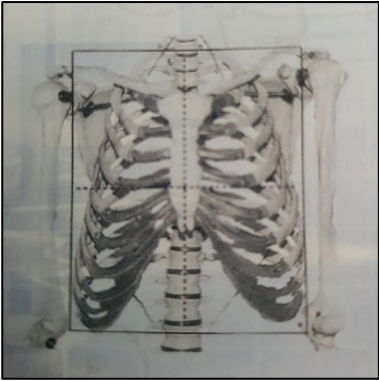
There is a special spring inside the REANIMATOR® arranged in such a way that the “tic” shows the correct pressure for cardiac massage. In some cases, it cannot be excluded that even when using the REANIMATOR® correctly, further complications or damage may occur (eg broken ribs). Although one of the main aims of the development of the REANIMATOR® has been to make it easy to use, just when situations of particular stress or shock occur, it can happen that the user feels overwhelmed with handling the REANIMATOR® and makes mistakes.

Other indications

The manufacturer reserves technical modifications to the product as a result of advances in it.

The REANIMATOR® is a class I sanitary device DIMDI DE/CA64/128442/01, GMDN 60841.

Symbols used on the basic white plate of the REANIMATOR®: CE, hourglass symbol, and (expiration date).

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